

# HIV FIND™

By

Meril

IVD **CE**  
2265

## Whole Blood

### HIV 1/2 Antibody Detection Self Test

**1** SINGLE USE  
HIV SCREENING TEST

**15 MINUTES**  
RESULTS

**HOME TEST**  
IN 4 EASY STEPS

**SINGLE PRICK**  
ONE DROP TEST



# Whole Blood

## HIV 1/2 Antibody Detection Self Test

### DO'S & DON'T'S

- ◆ For single use only. Open the foil pouch only when ready to test.
- ◆ Follow the User Guide carefully for accurate results.
- ◆ Sit in a clean, well-lit area and have all materials ready before starting.

### KIT CONTENTS



Test Device Pouch



Prefilled Assay Buffer



Lancets



Sample Dropper




Alcohol Swab



Cotton Gauze Pad



Bandage

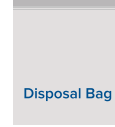

Disposal Bag



Instructions for use

### TEST PROCEDURE

**1**

Clean the finger with the alcohol swab and let it dry.

**2**

Press the lancet against finger until it clicks.

**3**

Hold the sample dropper tip to the blood and fill to the mark line.

**4**

Add the blood into the sample well of test device.

**5**

Twist off the ampoule top, hold it vertically, and add 3 drops of buffer to the specimen well.

**6**

Start the timer; read the test result at 15 mins.

**7**

**DISPOSE**

Place the test device, dropper, lancet, alcohol swab, gauze pad, and assay buffer ampoule in the disposal bag. Seal and discard the bag in the waste bin.

### RESULT INTERPRETATION

**POSITIVE RESULT**

If two lines appear even if faint, you tested HIV positive.

This is a screening test. Visit a clinic for further testing.

**NEGATIVE RESULT**

If one line appears at the C (Control Line) you tested HIV negative.

There must be no line at the T (Test line). This is a screening test. Test again after 3 Months.

**INVALID RESULT**

If no line appears at the T (Test line) & C (Control Line),

The test result is invalid, and if no line appears at C (Control Line) & Line appear at T (test Line), the test result is invalid.

**\*Note:** In case of invalid or inconclusive results, take a new test device and perform the test again.

**Disclaimer:** HIVFIND Whole Blood HIV 1/2 antibody detection self test is designed for personal use only. This is a screening test. A positive result should be confirmed by a healthcare professional through additional testing. This test does not replace professional medical advice, diagnosis, or treatment. If you have any concerns or symptoms, please consult a healthcare provider. Follow all instructions carefully for accurate results. Keep out of reach of children.

### ORDERING INFORMATION

PRODUCT CODE	MATERIAL DESCRIPTION	PACK SIZE
HIVWBS-01	HIV 1/2 ANTIBODY DETECTION SELF-TEST	01 TEST
HIVWBS-02	HIV 1/2 ANTIBODY DETECTION SELF-TEST	05 TEST
HIVWBS-03	HIV 1/2 ANTIBODY DETECTION SELF-TEST	10 TEST
HIVWBS-04	HIV 1/2 ANTIBODY DETECTION SELF-TEST	25 TEST



Scan QR code for detailed video instructions

WBRO/HIVWBS01/01  
DATE:17/02/2025



Meril Diagnostics Pvt. Ltd.

D1-D3, Second Floor, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

+91 9624025718 | +91 9429632124 | Email: mahendra.s@merillife.com | askinfo@merillife.com